Prodont
Holliger

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www.prodont-holliger.com

Summary :


1.	INTRODUCTION	9
2.	INSTALLATION AND SECURITY	10
3.	FUNCTIONING	12
4.	SERVICE	14
5.	ENVIRONMENT	15
6.	GUARANTEE	15
7.	SPARE PARTS	15
8.	TECHNICAL FEATURES	15
9.	MANUFACTURER INFORMATIONS	15

1. INTRODUCTION :

1.1. HOT AIR BURNER :

- The *AIRHOLY*[®] is used to heat instruments for sculpting wax and to soften waxes in dental prosthetic laboratories.
- The hot air burner *AIRHOLY*[®] is intended for professional use only.

1.2. DESCRIPTION :

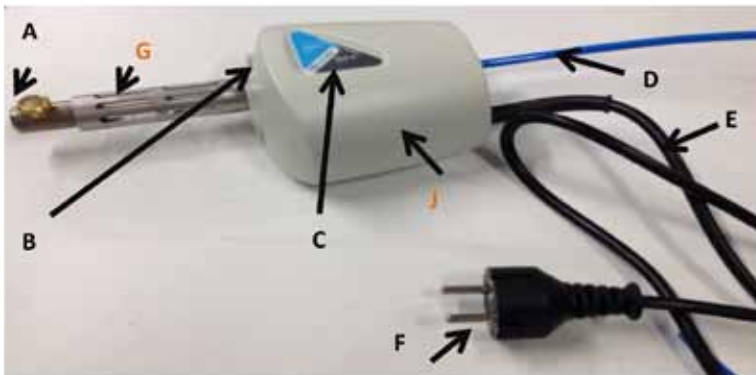Removable
hot air nozzle

- A Removable hot air nozzle
- B LED functioning light
- C ON/OFF switch
- D Hot air pipe supply
- E Power supply cable
- F Electrical plug
- G Burner (Stainless steel)
- H Pressure screw (Stainless steel)
- I Muffer
- J Plastic cover

2. INSTALLATION AND SECURITY

2.1. INSTALLATION AND CONNECTION :

- The hot air burner *AIRHOLY*[®] is intended for indoor use only and for an operating temperature between 10 and 30°C.
- It must be put on a stable workbench able to resist hot wax drippings.
- **The electrical connection of the device has to be made on an installation protected by a differential circuit breaker of 30 mA and by an electrical current protection of 10 A. For that purpose, use a 2 phases plug + T, 230VAC – 50 Hz – 10 Amp max.**
- The device must be placed away from the edge of the table to prevent any fall.
- The power supply cable (E) must be placed in order to prevent any contact with hot parts, especially the nozzle (A) and the burner (G).
- The device must be supplied with pressurized air. The laboratory must be equipped with a pressure reducer to control the air flow (sold separately).
- The air supplied must be dry and clean. An air full of particles or humidity can damage your electric installation and the device.
- Connect the burner's air supply pipe to the pressure reducer, the pressure must be set at 0.5 bar.

2.2. IMPORTANT SAFETY GUIDELINES

This device complies with CE regulations.

WARNING – To reduce burning, fire, electric shock or physical injuries hazards during use of the device, the following safety guidelines must be observed, including:

- Read the instructions carefully before using this professional device. Keep the instructions in an appropriate place. Please forward them with the device.
- You should never leave the hot air burner *AIRHOLY* unattended when plugged in. Unplug it immediately after use.
- Don't let anybody play with device.
- Use this device for its intended use, as described in the instructions. Use exclusively accessories approved by the manufacturer, as mentioned in the instructions.
- Do not use the device near flammable products.
- Do not use the device near sprays or when oxygen is in use.
- Do not unplug the device by pulling the power cable. Pull the electrical plug.
- Before use, check that the nozzle (A) is oriented upward and firmly attached to the burner with the pressure screw (H). Use a 1.5mm Allen wrench if needed.
- This device is designed to produce hot air; it comprises hot parts, in particular the nozzle (A) and the burner (G). Burning hazards pictograms are engraved on the device to identify the dangerous areas not to be touched to avoid severe burns.
- Do not remove the nozzle (A) until the device has cooled down.

A

3. OPERATION :

3.1. Before each use :

Make sure that the device is connected to the pressure reducer, and that the pressure reducer is connected to the air supply. Otherwise connect the device and make sure that the air pressure is set at 0.5 bar.

3.2. Operation of the hot air burner *AIRHOLY®*

- Plug the electrical plug (F) into the electrical power supply and make sure that the power cable (E) is directed away from the burner (G).
- Turn the device on by pressing the button ON on the control panel (C).

- At 20°C, with an air supply at 0.5 bar, the air temperature at the nozzle (G) reaches approximately 300°C.
- Outside the range of 0.5 / 0.4 bar, the device won't function properly. It's equipped with a security device (a thermistor).
If the air pressure diminishes gradually, the device will cease to function and will switch to fault mode. If the pressure is too strong, the air temperature will be too low for the intended use.
- The hot air burner is equipped with a red LED light (B) which indicates when the device is functioning.
- LED display modes:

Normal mode:

- ▶ continuous flash (the device is heating)
- ▶ 1 long flash (the device is cooling)

Fault mode :

- ▶ 2 short flashes (thermistor is on [lack of air]). **Quickly press the OFF button to stop the device. Check the air supply as described in chapter 2-1 to turn the device back on. If the problem occurs repeatedly, contact the customer service.**
- ▶ 1 short repeated flash (resistance out of service). **Press the OFF button to stop the device. It's damaged, contact the customer service.**

- To shut the device down, you must press the OFF button on the control panel (C). After pressing the button, the LED (B) will blink in the long flash mode and the device will continue to produce air for 2 minutes in order to cool down the burner and the nozzle.
- Don't unplug the device or cut the air supply to shut it down. It would damage the device.
- The burner will automatically shut down after 2 hours of continuous functioning, to prevent hazards of unattended functioning. If it happens during long use, press the ON button to turn the device back on.

4. SERVICE

- Cleaning

- Due to wax projections and depending on the way it's used, it may be necessary to clean the removable hot air nozzle (B). Operate on a cold and unplugged device. First unscrew the pressure screw (H) with an 1.5mm Allen wrench and remove the nozzle (B). The muffler (I) can also be removed for cleaning or replacement. Use a 14mm flat wrench.
- Don't let liquid fall in the nozzle (B) or burner (G).
- Don't jam the nozzle (B)
- Don't use solvents or chlorinated products to clean the plastic cover.

- Storage

- If you don't intend to use the hot air burner *AIRHOLY* for a long period of time, unplug the air supply and the power cable.
- Store the burner in a cool dry place, away from light..

Don't service or open the plastic cover (J) if the device is damaged or doesn't function properly. Contact the customer service.

5. ENVIRONMENT
The hot air burner *AIRHOLY*[®] contains electrical parts that complies with ROHS and DEEE.
6. GUARANTEE
The hot air burner *AIRHOLY*[®] has a 1 year guarantee.
7. SPARE PARTS
- Art. 1: Nozzle
 - Art. 2: Pressure screw M3
 - Art. 3: Muffler
8. TECHNICAL FEATURES

Physical features

Dimensions (mm)	260x110x70
Weight	640g
Cover material	Plastic (Polycarbonate)

The sound level in normal conditions is inferior to 54 dBA measured at 50cm with an air pressure at 0.4 bar.

Electrical features

Single-phase power supply 230 VAC – 50Hz with class 1 earth.

- Power 200 W – Intensity max : 1 Amp
- Distribution system neutral TT.
- Power surge II

9. MANUFACTURER INFORMATIONS

PRODONT HOLLIGER
La Marnasse
63880 OLLIERGUES (FRANCE)

Tel. +33(0)4 73 95 56 42
Fax +33(0)4 73 95 56 99
Email info@prodont-holliger.fr
www.prodont-holliger.com